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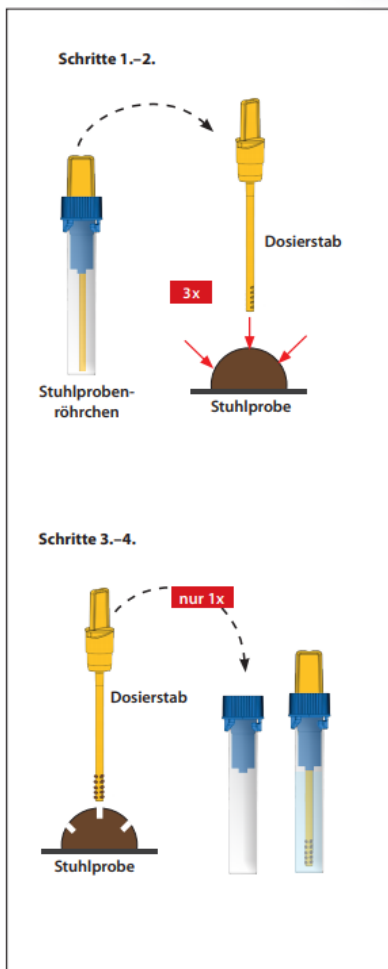
Stuhlprobenröhrchen IDK[®] TurbiTUBE[®]

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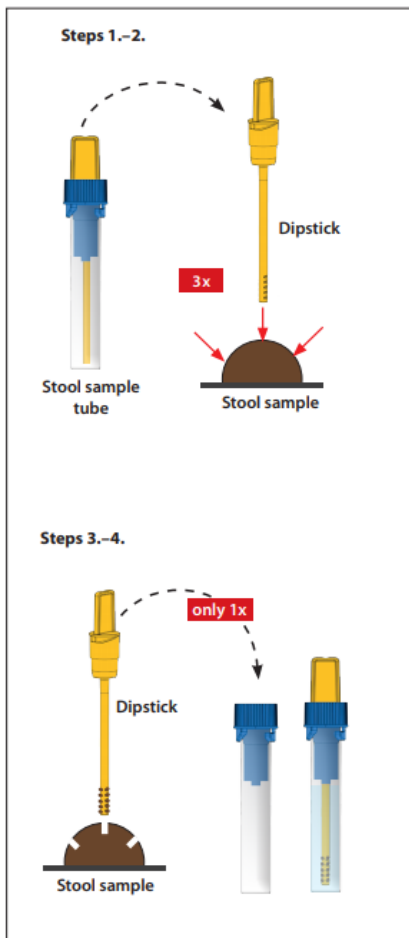
Instructions for sampling

stool sample tube
IDK[®] TurbiTUBE[®]

Application of the stool sample tube

1. Remove dipstick

Open the **upper yellow part** of the cap and remove the attached dipstick. The blue cone-shaped insert remains on the tube!

2. Obtain stool sample

Insert the dipstick into the stool at 3 different sites. Make sure that all notches at the lower part of the dipstick are covered completely with stool.

3. Place dipstick back into the tube

Stick the dipstick with the attached stool **only once** back into the sample tube. Excess material will be stripped off by the cone-shaped insert, letting 15 mg of sample into the tube.

4. Prepare stool sample suspension

Close the tube properly by turning until you hear the yellow shutter click. Then shake well until no stool sample remains in the notches and the stool is completely suspended in the extraction buffer. The suspension now contains 15 mg of stool

Attention: The **stool dilution** in the sample tube **hardly changes colour**. Repeated insertion of sample into the tube will falsify the results!

Do you have questions? Our employees will be happy to help you.
Contact us by phone or email!

+43 (0) 2742 238 38

Mathilde Beyerknecht-Straße 9

3104 St. Pölten

hotline@enml.at

For booking an appointment
at the ENML laboratory
institute
Scan the QR code!

https://play.google.com/store/apps/details?id=tw.mobileapp.qrcode.banner&hl=de_AT&gl=US